TRACOE tube clean Cleaning Tub

TRACOE tube clean Reinigungsdose

REF 934

EN Instructions for Use

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SV Bruksanvisning

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DA Brugsanvisning

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NO Bruksanvisning

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EN / Please read Instructions for Use • **DE** / Gebrauchsanweisung beachten • **NL** / Rekening houden met de gebruiksaanwijzing • **FR** / Respecter le mode d'emploi • **IT** / Consultare le istruzioni per l'uso • **ES** / Siga las instrucciones de uso • **PT** / Ter atenção às instruções de utilização • **SV** / Se bruksanvisningen • **DA** / Følg brugsanvisningen • **NO** / Følg bruksanvisningen • **FI** / Noudata käyttöohjetta • **PL** / Zazrzyj do instrukcji używania •

MD

EN / Medical Device • **DE** / Medizinprodukt • **NL** / Medisch hulpmiddel • **FR** / Dispositif médical • **IT** / Dispositivo medico • **ES** / Dispositivo médico • **PT** / Dispositivo médico • **SV** / Medicinteknisk produkt • **DA** / Medicinsk udstyr • **NO** / Medisinsk enhet • **FI** / Lääkinnällinen laite • **PL** / Wyrób medyczny •

REF

EN / Item number • **DE** / Artikelnummer • **NL** / Artikelnummer • **FR** / Référence • **IT** / Codice prodotto • **ES** / Número de artículo • **PT** / Número do artigo • **SV** / Katalognummer • **DA** / Artikelnummer • **NO** / Artikkelnnummer • **FI** / Tuotenumero • **PL** / Numer katalogowy •

EN / Batch code · **DE** / Chargencode · **NL** / Lotcode
· **FR** / Code de lot · **IT** / Numero di lotto · **ES** / Código
de lote · **PT** / Código do lote · **SV** / Satsnummer · **DA** /
Batchkode · **NO** / Chargekode · **FI** / Eräkoodi · **PL** / Kod partii ·

EN / Manufacturer · **DE** / Hersteller · **NL** / Fabrikant ·
FR / Fabricant · **IT** / Produttore · **ES** / Fabricante · **PT**
/ Fabricante · **SV** / Tillverkare · **DA** / Producent · **NO** / Produsent ·
FI / Valmistaja · **PL** / Wytwórca ·

EN / Date of manufacture · **DE** / Herstellungsdatum ·
NL / Fabricagedatum · **FR** / Date de fabrication · **IT** /
Data di produzione · **ES** / Fecha de fabricación · **PT** / Data de
fabricação · **PT** / Data de fabricação · **SV** / Tillverkningsdatum ·
DA / Produktionsdato · **NO** / Produksjonsdato · **FI** / Valmistus-
päivä · **PL** / Data produkcji ·

EN / Single Patient – multiple use • **DE** / Einzelner Patient – mehrfach anwendbar • **NL** / Eén patiënt - meervoudig gebruik • **FR** / Un seul patient – à usage multiple • **IT** / Singolo paziente – uso multiplo • **ES** / Un solo paciente – uso múltipleo • **PT** / Paciente único - várias utilizações • **SV** / En patient - flera användningar • **DA** / Enkelt patient - flegangsbrug • **NO** / Kun til bruk på én pasient • **FI** / Potilaskohtainen- voidaan käyttää useita kertoja • **PL** / Wielokrotne użycie ujednego pacjenta •

EN / Store in a dry place • **DE** / Trocken aufbewahren • **NL** / Droog bewaren • **FR** / Conserver au sec • **IT** / Conservare in luogo asciutto • **ES** / Conservar en un lugar seco • **PT** / Guardar em local seco • **SV** / Förvaras torrt • **DA** / Opbevares tørt • **NO** / Oppbevares på et tørt sted • **FI** / Säilytettävä kuivassa • **PL** / Chronić przed wilgocią •

EN / Protect from sunlight • **DE** / Von Sonnenlicht fernhalten • **NL** / Beschermen tegen zonlicht • **FR** / Protéger de l'ensoleillement • **IT** / Non esporre alla luce del sole • **ES** / Mantener alejado de la luz solar • **PT** / Manter afastado da luz solar • **SV** / Skyddas från solljus • **DA** / Beskyttes mod sollys • **NO** / Må holdes unna sollys • **FI** / Säilytettävä auringonvalolta suojattuna • **PL** / Trzymać z dala od światła słonecznego •

Instructions for Use for TRACOE tube clean Cleaning Tub

TRACOE tube clean Cleaning Tub with an inner mesh basket is made of polypropylene and has a capacity of 1000 ml. It is indicated to be used with cleaning solution to clean tracheostomy tubes and placeholders.

1. Functional Description

Caution:

- When the product is used together with other medical devices, follow the respective instructions for use.
- Only one tube, i. e. one outer cannula together with the corresponding inner cannula(s) must be placed next to each other in the mesh basket at the same time to avoid a mix-up when reassembling the tube afterwards.

1. Prepare the cleaning solution (e. g. TRACOE tube clean Cleaning Powder (REF 932) or TRACOE tube clean Concentrated Cleaning Liquid (REF 933)) according to their instructions for use.

2. Use the mesh basket to place the product in the cleaning solution.
3. Cleaning steps for medical devices are described in their respective instructions for use.

2. Care and Cleaning

The product is for single patient use only.

If necessary, rinse the tub again with fresh cleaning solution.

Rinse the tub with potable water and dry it off.

The cleaning tub including the mesh basket can be autoclaved using a sterilization process with saturated steam (parameters: one cycle, 20 minutes, 121 °C, 1 bar).

Caution: The printing is not dishwasher safe.

3. Disposal

Used products are to be disposed of in accordance with valid national regulations, waste management plans, or clinical procedures governing biohazardous waste materials.

The cleaned tub can be disposed of as recyclable waste according to the local regulations.

4. Returns and Complaints

Returned products, that have been used, will only be accepted, if TRACOE has agreed to the return and a completed decontamination certificate and complaint report is enclosed. These forms are available either directly from TRACOE medical, or via the website www.tracoe.com.

If the device is involved in a reportable incident, as defined in local medical device legislation, please contact TRACOE medical (complaints@tracoe.com), and the appropriate regulatory body in the country of use.

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TRACOE medical GmbH